



Software Installation Instructions

The latest software update is available at this website to download at any time:

<https://www.dropbox.com/s/ckggm7rx4opecco/ClearView5.0.9.iso?dl=0>

Before trying to update or install the software, make sure you go into the *Control Panel* on your computer and uninstall or remove all previous versions and drivers.

Do NOT have the camera plugged in during this process.

- From the link above, it will say *“.iso files can't be previewed”*, click on *'Download'* and choose an option. If you do not have a drop box account, click *'No Thanks, continue to download'*
- Open the *'ClearView5.0.9 zip file'* .
- Double click on the *'setup.exe'* Application. (Do not open the *' .msi '* Clearview WPF Setup-Windows Installer Package which would only install the app and not the drivers.)
- Continue through the install screens that will bring up the *“Welcome to the ClearView Setup Wizard”* install window. Proceed with the *“Device Driver Installation Wizard”* window that appears.
- After completion, it will inform you that it was successful and you can now connect your camera to the computer.
- Once the camera is plugged into the USB port, you may be informed that it is *“installing the device driver software”*. This may take a few minutes and then will inform you it has been installed correctly.
- You may now open up the *Optibrand ClearView software* icon on your desktop and begin using the camera.